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#### Dear Customer

Thank you for choosing this high-quality product from Dentaurum.

In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use.

The written instructions cannot cover all possible situations that can occur during use. For this reason, our hotline is available to answer any other questions or concerns that may arise.

Due to constant developments, we recommend, even when using the same products regularly, that you study the enclosed current instructions for use or refer to our website at [www.dentaurum.de](http://www.dentaurum.de).

#### 1. Manufacturer

Dentaurum J.P. Winkelstroeter KG  
Turnstraße 31 · 75228 Ispringen · Germany

#### 2. General product description

The latest development is the Herbst TS – a three-stage telescoping mechanism design. The telescoping design provides several key treatment advantages:

- an integral three-stage design which does not separate eliminating the risk of rod to sleeve disengagement during treatment.
- the lower screw and pivot (F-base) are positioned central to the second bicuspid and the upper screw and pivot are positioned central to the first molar, thus reducing tissue irritation.
- the use of speciality screws and F-bases allow for a dramatic increase in the ability of the patient to realize lateral movement of the lower jaw.

The Herbst TS is available in three lengths; 20 mm, 22 mm and 24 mm. The length is measured from the center of the upper eyelet to the center of the lower eyelet. Remember with traditional Herbst appliances you generally reduce the length of the sleeve and/or rod to correspond to the patient's specific requirements, with the Herbst TS no length reduction is required.

With the telescoping mechanism, the laboratory technician will position the lower pivot central to the second bicuspid, and place the upper pivot in the center of the first molar. These key placement points provides easier operator access to the screws, and reduces the overall bulk of the appliance once seated in the patients mouth.

The upper and lower eyelets have been manufactured with an offset; this design feature provides greater patient comfort since the head of the screw is in-line with the rod of the telescoping mechanism.

Since the Herbst TS does not separate to allow traditional spacer rings to slip over the Herbst rod, to continue mandibular advancement during the course of treatment just place and crimp our slotted spacer rings on the lower rod. This technique is easier and faster than the traditional method. Crimpable spacer rings and clamping pliers for Herbst TS are available.

#### 3. Fabricating and applying the Herbst TS appliance

Pour the impressions, which include the waxed out bands fitted on the 1st premolars and molars, with dental stone (e.g. RapiDür®). Mount the models in the correct position in an articulator with plaster using the activating bite recorded by the operator. Bend and solder (or laser weld) the half-round remaloy® wire (ø 0.75 x 1.5 mm) or round remaloy® wire (ø 1.0 mm) to the bands to form a lingual arch. A palatal bar can be placed in the upper jaw between the first molars to provide stabilization.

Next spot weld and solder the bases of the hinge units to the bands (e.g. using the Master 3000) or laser welded. Ensure that the screws are at the correct angle to the axis between the upper and lower jaw. It is also important that the hinges between the upper and lower jaw operate without friction to ensure smooth functioning of the telescopic mechanism. Friction can damage the appliance – especially the bands.

The telescope tubes and telescope rods cannot be adjusted individually with the Herbst TS retention hinge. The matching retention hinge is selected according to the activating bite by closing it and placing it on the base or by measuring the distance between the two bands beforehand.

##### Notes:

- “Double bands” should be used to minimize the risk of the bands being damaged by shear forces. This involves welding a second band over the fitting band (e.g. using the Master 3000). Generally the second band is 2 to 3 sizes larger than the first band fitted.
- The loops of the Herbst TS must not be bent!

##### Fitting intraorally:

The bands are fitted intraorally using standard band cement (e.g. GlasTec). We recommend covering the bases of the hinges with wax to ensure that the cement does not penetrate into the thread. After curing of the cement the Herbst TS is unscrewed.

If necessary, the Herbst TS appliance can be activated using various spacers (see Product range).

Make sure that the appliance can be removed quickly in case of a medical emergency, if necessary.

#### 4. Composition

The composition is included in the materials list. Refer to the catalog or [www.dentaurum.de](http://www.dentaurum.de).

#### 5. Safety instructions

Patients known to be allergic to one or several substances used in this product must not be treated with this product. The fit of the Herbst TS hinge and tightness of the screws should be checked at regular intervals to prevent the appliance spontaneously coming apart and the components being swallowed or aspirated! If necessary, tighten the screws and secure them with a special screw securing adhesive, e.g. Ceka® Bond. The loops of the Herbst TS must not be bent. It is not intended that the loops can be ground.

#### 6. Information on single-use products

The Herbst TS retention hinge is intended for single use only. Reconditioning of Herbst TS that has already been used (recycling) and its reuse on a patient is not permitted.

#### 7. Product range

##### Herbst TS retention hinge

Telescopis system

For welding or soldering to bands or cast splints.

**Length: 20 mm**      **REF 607-126-20**      1 Set

**Length: 22 mm**      **REF 607-126-22**      1 Set

**Length: 24 mm**      **REF 607-126-24**      1 Set

Contents:

- 1 x Telescope attachment for right
- 1 x Telescope attachment for left
- 4 x F base
- 4 x Allen screw
- 1 x Instructions for use

The hexagon socket screw key is not included in delivery.

**Hexagon socket screw key**      **REF 607-144-00**      1 piece

for Herbst TS/I/II/SUS<sup>2</sup>

sterilizierbar

wrench size 1.5

**Herbst F base**      **REF 607-127-30**      10 pieces

for Herbst TS + I retention hinges

##### Herbst TS spacers, slotted

**Length: 1 mm**      **REF 607-103-11**      10 pieces

**Length: 2 mm**      **REF 607-104-11**      10 pieces

**Length: 3 mm**      **REF 607-105-11**      10 pieces

**Length: 4 mm**      **REF 607-107-11**      10 pieces

**Herbst TS clamping pliers**      **REF 003-710-00**      1 piece

sterilizierbar

For clamping slotted spacers for the Herbst TS retention hinge.

Premium Line – 10-year guarantee

#### 8. With reference to our quality standards

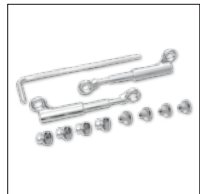
Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

#### 9. Explanation of symbols used on the label

REF	Order number
 12345	Batch description
 0483	CE Mark MP Class 2
 → <a href="http://www.dentaurum.de">www.dentaurum.de</a>	Instructions for use available in the internet
	Not for reuse
	Codablock barcode
	Matrix code
	Green dot
Date of Information: 08/09	
Subject to change	

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